
URINE COLLECTION INSTRUCTIONS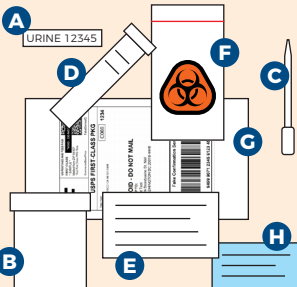

Included in this kit


- A** Self-adhesive order number label
- B** Urine cup
- C** Bulb pipette
- D** Clear-capped transport tube with fluid
- E** Lab Test Requisition Form
- F** Zip closure biohazard bag
- G** Postage-paid return envelope
- H** Instructions for results

IMPORTANT: Read all instructions prior to collecting your sample

- Use the cup to collect your first urine of the day.
- Be sure to wash and dry your hands thoroughly, **before** and **after** collection.


START HERE

1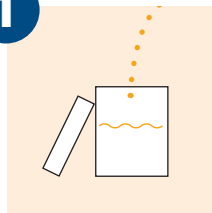


Remove the contents of the plastic bag.

Use the cup to collect your first urine of the day, filling at least half of the cup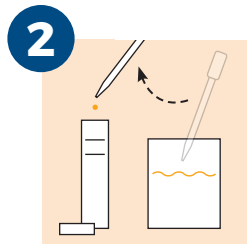


Set the clear-capped tube onto a flat, solid surface and remove the cap.

Using the bulb pipette, transfer urine from the cup into tube of fluid until the liquid reaches the black fill lines.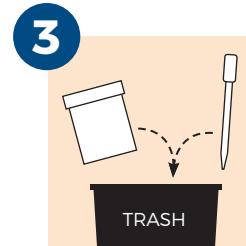


Discard the remaining urine, bulb pipette, and the cup.

! Do not mail the urine cup.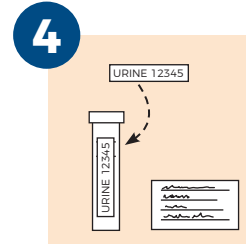

Place the self-adhesive label with your order number on the clear-capped tube. Complete the Lab Test Requisition Form with your name, date of birth, gender, and collection date and time.

! If you are testing multiple parts of your body, ensure that you are placing the correct label on each tube. The labels are letter-coded and marked to correspond with the part of your body being tested.

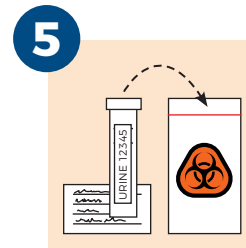Tightly seal the labeled transport tube – lid up – and Lab Test Requisition Form in zip closure **ORANGE BIOHAZARD BAG** with absorbent, then place bag into clear plastic bag and then into postage paid mailer and seal tightly.

! If any of these steps are missed or not preformed correctly, specimens will be discarded and no results will be provided.

You can send your sample from your mailbox or any United States Post Office.

! Do not place in a FedEx or UPS drop box

To receive your results, log into your My SNHD account at www.snhd.info/c2p 7-10 business days after mailing the collection kit. If you receive a phone call, email, or text from a Southern Nevada Health District staff person, it is very important for you to respond. Results will not be mailed at this current time.

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